

Mylan HIV Self Test

is the world's only integrated
HIV Self Test device¹

WHO
Prequalified²



Safe¹



Accurate¹



Convenient¹




- ✓ Removes need for multiple test components¹
- ✓ Reduces user errors¹
- ✓ Simplifies test procedures¹
- ✓ Improves test accuracy¹

Mylan HIV Self Test is Easy to Use

Step 1

Push grey button in to prick finger integrated lancet!
Push hard

IMPORTANT
It only pricks once!

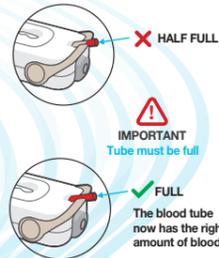

Step 2

Fill blood tube with blood. If blood tube isn't full, squeeze finger and add more blood.¹

Touch blood to tip of blood tube

Step 3

Hold test on table. Now flip blood tube over to the well.¹

IMPORTANT
Check blood has moved from tube to well

Step 4

Add 4 drops of test fluid in the well.¹

IMPORTANT
4 drops

Step 5

Wait 15 minutes before reading the result. Do not wait more than 20 minutes.¹

IMPORTANT

⊖ HIV negative

If one line appears at the C, you tested HIV negative. There must be no line at the T.

This is a screening test.

Test again after 3 months.

See included care card for additional information.




? Test did not work

If no line appears at the C, the test did not work.

This is a screening test. Go to a clinic for further testing.



See included care card for additional information

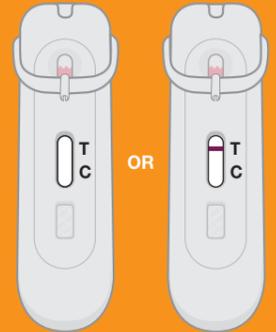


+ HIV positive

If two lines appear, even if faint, you tested HIV positive.

This is a screening test. Go to a clinic for further testing.



See included care card for additional information

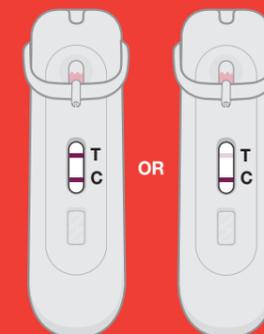

For more information and to watch step by step test procedure video please visit : <https://www.mylanez2uz.com>

Important Note: Wash and dry hands before performing the test. Instructions for use that comes with the pack should be read carefully before performing the test. This is a screening test. If your result interprets as "HIV negative", please test after 3 months. If result interprets as "Test did not work" or "HIV Positive", go to a clinic for further testing.

Mylan HIV Self Test

is recognized for innovative design*



A

Built-in Safety Lancet¹

The auto-retracting safety lancet eliminates the risk of hazardous sharps injuries by locking the needle inside the device after use for safe disposal.

B

Accurate Blood Collection¹

The easy-to-use blood collection unit is designed to collect exactly the right sample volume (10µl), ensuring optimal test performance.

C

Easy Blood Delivery¹

Blood is delivered cleanly to the test strip by simply rotating the blood collection unit.

D

Easy Interpretation of Result¹

The test result is displayed clearly in just 15 minutes.

*1



References:

1. Data on file.

2. WHO PQ Public Report ; Version 1.0; July 2019.

Available at: https://www.who.int/diagnostics_laboratory/evaluations/pq-list/190708_pqdx_0320_090_00_pqpr_mylan_hiv_self_test.pdf; Accessed on 1st April 2020

For further information, please contact:

Mylan Pharmaceuticals Private Limited

No. 32/1 & 2, 34/1 to 4, 7th to 12th Floor,
Prestige Platina Block 3, Kadubeesanahalli Village,
Varthur Hobli, Outer Ring Road, Bangalore East Taluk,
Bangalore – 560 103.

Tel: +91 80 6672 8000

Fax: +91 80 6672 8899

E – mail: mylan.india@mylan.in

Website: www.mylan.in

Registered Office

Plot No. 1-A/2, MIDC, Industrial Area,
Taloja, Panvel, Raigad (Dist.),
Maharashtra – 410 208, India
CIN: U73100MH1997PTC106267

For more information please visit:

<http://www.ez2uzhivselftest.com>



Follow us on:



#Ez2UzHIVSelfTest

